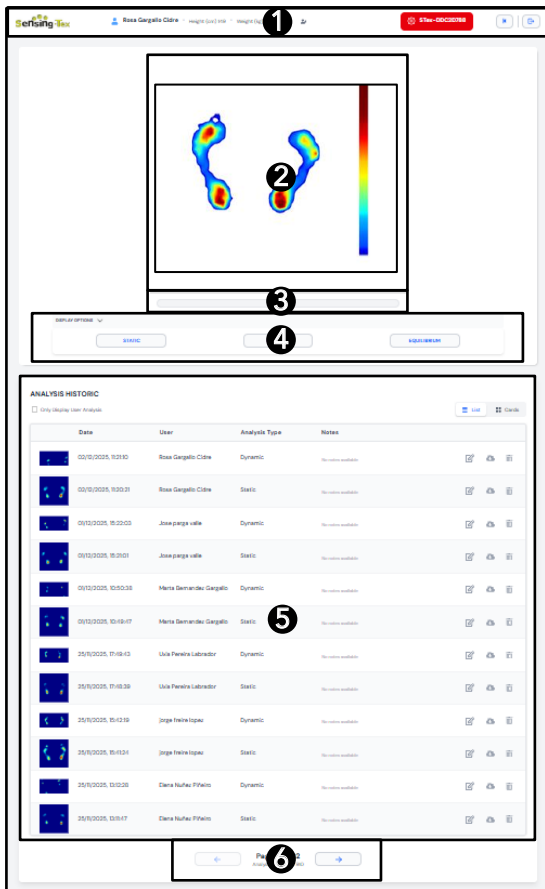
# QUICK USER GUIDE

## 1. PATIENT AND CLINIC DATA CONFIGURATION

- **Left:** Patient Management. You can edit a patient's profile or create a new one.
- **Right:** A colored button shows platform status (Red/Inactive, Green/Active). Click to edit company name, address and logo.

## 2. VISUALIZATION OF PRESSURE DATA

This screen displays real-time pressure mapping data.

## 3. LOAD BAR

The progress bar shows the analysis progress. When the bar is full, the analysis is completed.

## 4. ANALYSES

Select the analysis type, then click the button to begin.

## 5. ANALYSES HISTORY

Once the analyses are completed, they will be saved in this section. You can consult them whenever you need to. You can also download them and add comments.

## 6. MANAGE SAVED ANALYSES

This section displays the results of 12 analyses. You can switch pages using the buttons to view previous analyses.

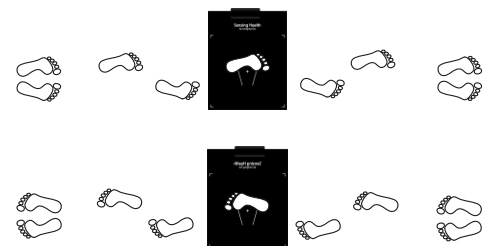
## A. STATIC ANALISYS

- Position the patient on the platform with their feet centered.
- Click the button to begin the analysis. Read the Tutorial and Click Ready.
- Wait for the loading bar to reach 100%.

## B. DYNAMIC ANALISYS

- Position the patient one meter and a half away on the left side of the platform.
- Click the button to begin the analysis. Read the tutorial and Click Ready. (you will have **20 seconds** to step two times left and right following this sequence):
  - The patient walks toward the platform, placing their left foot on it (they must take two steps before reaching the platform).
  - The patient turns 180 degrees.
  - The patient walks toward the platform, placing their right foot on it (they must take two steps before reaching the platform).


## C. EQUILIBRIUM ANALISYS

- Position the patient on the platform with their feet centered.
- Click the button to begin the analysis.
- The analysis duration is **20 seconds**, but the data is collected on the first **10 seconds**.
- Wait for the loading bar to reach 100%.

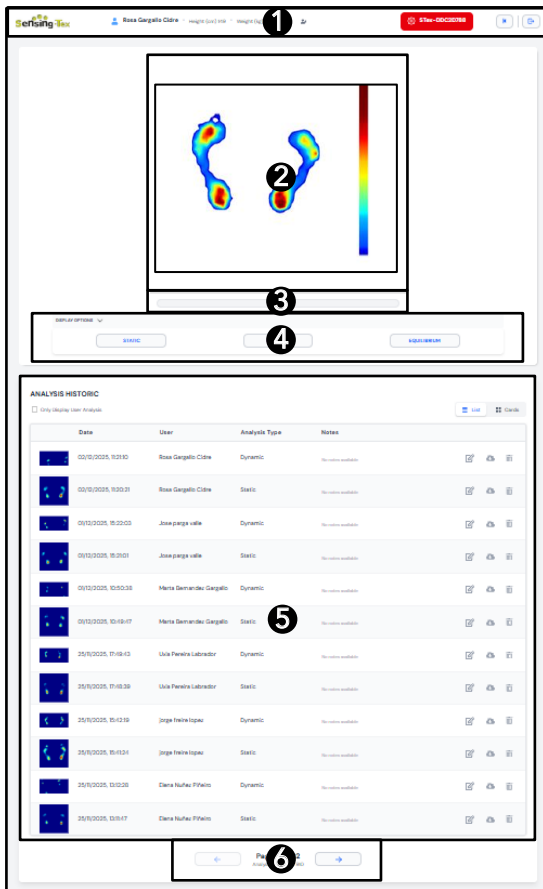For detailed information on the functionality of the members' area, please refer to the User Manual provided by your distributor. The platform is not recommended for use by children.

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